



Active Ice® Quick Reference

- 1. Fill cooler with ice.***
- 2. Fill cooler with water to the line indicated (always be sure that water covers the pump).***
- 3. Before first use of the unit, after connecting the bladder and filling the unit with ice and water, lay the bladder on a flat surface & run the unit for 5 minutes to stabilize. Do not attach the bladder to the body during this time. This will allow the bladder to fully engorge with water and help prevent kinks.***
- 4. Always be sure that there is a return flow of water from the top return flow tube.***

READ ALL WARNINGS AND CAUTIONS BEFORE SYSTEM OPERATION.

THE ACTIVE ICE® SYSTEM IS MANUFACTURED BY POLAR PRODUCTS, INC. PLEASE CONTACT US DIRECTLY AT 1.800.763.8423 IF YOU HAVE ANY QUESTIONS.

Polar Products, Inc. www.polarproducts.com

