

LITHIUM-ION BATTERY PACK INSTRUCTIONS



Battery Pack comes with its charger. It should arrive fully charged. The battery pack has two cords. One is labeled “To Cooler” and the other “To Charger.” Use the “To Cooler” cord to plug into the cooling reservoir to run the unit off the battery pack. To charge the battery pack, unplug the battery pack from the unit, plug the cord labeled “To Charger” into the charger and plug into a wall outlet. Only one cord is in use at any time.

When unit is charging, a red light is shown. Once battery pack is charged, a green light will show. The battery pack will run the unit for approximately 6 hours.

Please Be Advised: Never use the battery pack in conjunction with the 12-volt transformer, only use with the included charger. Never attempt to charge the battery pack and run the unit at the same time.

polarproducts.com | 800.763.8423 | polar@polarproducts.com