

Dual Use Addendum

Your cooler unit has been modified so that it can work both with the chiller unit and, with ice, without the chiller unit. The cooler has two on/off switches, one for operation of each pump inside the cooler. One pump circulates water into the chiller (where it is chilled down to the set temp) and back into the cooler. The second pump circulates water through the head cap, vest, etc. When using with the chiller, it will need power from a standard wall outlet, 120 volt AC. When using without the chiller, the cooler can be run with the battery pack or the power take off because all it needs is 12 volt DC. Ice will also need to be added to the water since it is not being chilled.

To Use With Chiller: Turn both on/off switches to on. This will turn on both pumps. Fill the unit with enough water to submerge both pumps. No ice necessary.

To Use Without Chiller: Only turn on the switch that is labeled “cooler.” Fill the unit with enough water to submerge the pump, then fill to the return tube/ fill line with ice.