


Solutions for Cooling Every Body®

Polar manufactures all major cooling technologies!

We can help you find the best value and most effective products for your specific application and climate.

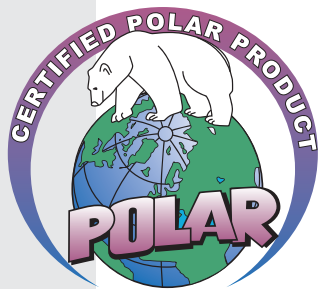

Kool Max™

Polar's best-selling choice for personal cooling!

Kool Max™

Frozen Water-Based Cooling Packs



Cool Comfort®

Water-Activated / Frozen Hybrid Cooling



Quick Cool™

Soft Frozen Packs for Short-term Cooling



Cool58™

58° F Phase Change Cooling Packs



Cool Flow®

Circulating Cold Water System



Evaporative

Water-Activated Cooling




See our complete cooling product lines at www.polarproducts.com.

Also available from Polar Products:



HOT & COLD Therapy

- **Active Ice® Cold Therapy System**
Ideal for post-surgery pain relief.
- **Soft Ice® Hot/Cold Packs & Compression Wraps**
Keep on hand for acute injury and swelling.
- **Thera-Temp® Moist Heat Therapy Wraps**
Alternate with cold therapy for chronic pain.
- **Roller Ice® Trigger Point Massage Tools**
Convenient and comfortable. No mess!

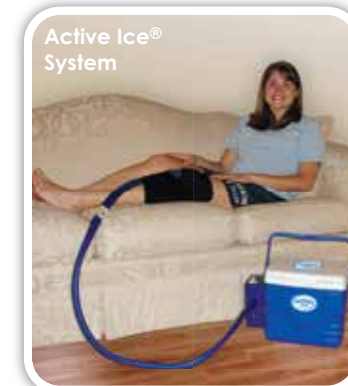


Active Ice® System



Soft Ice® Compression



Thera-Temp® Moist Heat

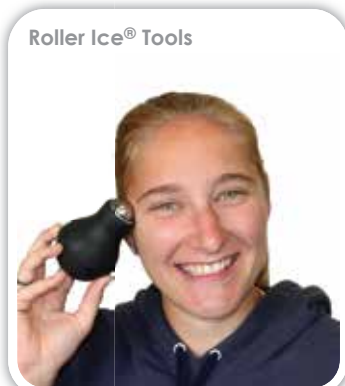

Roller Ice® Tools

Visit our website to see all of our therapy products.



How does it work?

Frozen water-based Kool Max™ Cooling Packs fit into insulated pockets in vests and accessories. Lightweight and efficient packs cool the body in any environment.

CARE & USE MANUAL

The Kool Max™ Advantages:

Lightweight and Versatile

Kool Max™ vests and wraps are designed to deliver effective cooling with minimal weight. You control the weight and location of cooling by adjusting the number and location of the packs used in your vest.

Quality Materials

Kool Max™ garments are designed and constructed for long term performance. They will last for many years with proper care, maintenance and storage.

Highest Cooling Energy

The water-based technology provides the highest cooling energy of any cooling pack. Kool Max™ requires the use of a freezer, but the packs return this energy in cooling. It's science!

Improves performance, safety and comfort

Kool Max™ lessens the debilitating effect of heat during work, athletics and day-to-day life for those with heat sensitivity.

Extended Cooling Relief

Cooling for up to 3.5 hours in any climate. Cooling time varies depending on many factors, including activity and environment.

Activation and Use Instructions

If you have any questions or concerns regarding your product, please contact our Customer Service at **1.800.763.8423** or send an e-mail to polar@polarproducts.com. We will be happy to help!

Using the Kool Max™ Cooling Packs

Our reusable Kool Max™ Cooling Packs are water-based, non-toxic and non-flammable. Place the packs individually and flat into a freezer until frozen hard. For best results, store the packs in the freezer, so they're ready when you are. When ready to use, simply remove the packs from the freezer and place in the insulated vest or wrap pockets. Frozen packs may be transported or stored in a cooler for later use.

Kool Max™ Cooling Packs



Wearing the Kool Max™ Vest and Wraps

All Kool Max™ vests and wraps should fit snug to the body for optimal performance. Many of our Kool Max™ garments provide adjustable fit through the use of belts and straps with hook and loop tabs. Simply place the garment on your body and adjust straps for a snug fit.

Cooling and Weight Adjustment

The amount and location of cooling may be controlled in many Kool Max™ vests and wraps by adjusting the number and location of Kool Max™ packs used. The cooling potential may also be adjusted by wearing one or more layers of clothing between the body and the garment. **Note: one or more layers of insulating fabric should always be worn between the skin and the cooling pack.**

Maintenance and Care

Vest & Wraps:

Hand wash in cool water with a mild detergent.
Hang in a well-ventilated area to dry.

Kool Max™ Packs:

Packs may be wiped clean with a damp cloth. **Note: Store packs in freezer or cool, dry environment when not in use.**

Cooling Duration

The length of time Kool Max™ cooling vests and wraps will cool depends on many factors, including the activity and body temperature of the person using the items and the temperature of the environment. Additionally, the size and number of packs used will vary the amount of cooling.

Cautions and Warnings: Consult your health care professional prior to using this or any cooling products for a medical condition. Kool Max™ Cooling Products are designed for the specific objective of cooling the body. Polar Products recommends that all cooling products be worn over clothing or other fabric layers to regulate the products' cooling potential. The amount of layering required to ensure the highest level of cooling and comfort will vary from person to person and is strictly the responsibility of each individual user. **Polar Products, Inc. will not be liable for injuries or damages resulting from misuse or misapplication of our products.**

Kool Max™ Vests & Accessories

Full coverage Kool Max™ cooling vests:



Discreet torso cooling vests:



Save money with cooling kits:



Targeted cooling accessories:



All styles of neck wraps:

