



Kool Max[®]

Secrets Torso Wrap

Frozen water-based Kool Max[®] Cooling Packs fit into insulated pockets in the torso wrap. Four lightweight, reusable packs cool the body in any environment for up to two hours. This discreet wrap can be worn beneath clothes and during physical activity!

KMS



Secrets Torso Wrap Includes:

- Four 4.5" x 6" Kool Max[®] Packs
- Insulated torso wrap with two attached elastic belts with hook closure tabs

(Please note: to adjust for a snug fit, closure tabs can be connected anywhere on the front of the wrap.)

Great for pre-cooling, post-cooling, and cooling during activity.

Kool Max[®] lessens the debilitating effect of heat during work, athletics and day-to-day life for those with heat sensitivity.



POLAR Products

www.polarproducts.com

1.800.763.8423

Directions for Use:

Using the Kool Max® Cooling Packs

Our reusable Kool Max® Cooling Packs are water-based, non-toxic and non-flammable. Place the packs individually and flat into a freezer until frozen hard. For best results, store the packs in the freezer, so they're ready when you are. When ready to use, simply remove the packs from the freezer and place in the insulated wrap pockets. Frozen packs may be transported or stored in a cooler for later use.

Wearing the Kool Max® Secrets Torso Wrap

The Kool Max® Secrets Torso Wrap should fit snug to the body for optimal performance. The elastic straps are attached to the back of the wrap. The hook closure tabs on the ends of the elastic straps will attach anywhere on the front of the wrap. Attach one of the elastic straps to the front of the wrap, wrap the torso wrap around your waist, and attach the other strap to the front of the wrap on the other side. Adjust elastic strap attachment for a snug, comfortable fit.

Cooling and Weight Adjustment

The amount of cooling and total wrap weight may be controlled by adjusting the number of Kool Max® packs used. The cooling potential may also be adjusted by wearing one or more layers of clothing between the body and the garment.

Note: one or more layers of insulating fabric should always be worn between the skin and the cooling pack.

Care and Storage:

Secrets Wrap: Hand wash in cool water with a mild detergent. Hang in a well-ventilated area to dry.

Kool Max® Packs: Packs may be wiped clean with a damp cloth.

Note: Store packs in freezer or cool, dry environment when not in use.

Warnings and Cautions:

Consult your health care professional prior to using this or any cooling products for a medical condition. Kool Max® Cooling Products are designed for the specific objective of cooling the body. Polar Products recommends that all cooling products be worn over clothing or other fabric layers to regulate the products' cooling potential. The amount of layering required to ensure the highest level of cooling and comfort will vary from person to person and is strictly the responsibility of each individual user. **Polar Products Inc. will not be liable for injuries or damages resulting from misuse or misapplication of our products.**