



ACTIVE ICE® 3.0 COLD THERAPY SYSTEM DRAIN ACCESSORY

A Drain Accessory is available to add to all Active Ice® 3.0 Cold Therapy Systems. It allows the user to empty most of the water from the cooling reservoir without lifting it. The Drain Accessory is conveniently clipped into the underside of the cooling reservoir lid.

To use, disconnect your therapy pad, and connect the Drain Accessory to one of the female couplings on the reservoir's tubing.

Then, place the Drain Accessory in an area where water may be drained without incident or injury.

Turn your Active Ice® System on and the water inside the cooling reservoir will be pumped out the end of the Drain Accessory.

If no water is flowing, connect the Drain Accessory to the other female coupling and repeat the above steps.

Let your Active Ice® System run until the majority of the water inside the reservoir has been removed, then turn off the system.

NOTE: THE DRAIN ACCESSORY WILL NOT DRAIN ALL WATER FROM THE COOLING RESERVOIR. IF THE PUMP IS LEFT RUNNING WITHOUT WATER FOR TOO LONG, IT MAY BURN OUT.

polarproducts.com | 800.763.8423 | polar@polarproducts.com

