

Polar Life Pod™

WARNINGS AND CAUTIONS

1. Read all instructions and warnings before using this product.
2. This product has not been sterilized. Always take proper precautions when using in situations where infection could occur due to open wounds or other circumstances.
3. Always make sure there is adequate water and ice available for use with the Polar Life Pod™.
4. Be sure all personnel are trained in the proper use of the Polar Life Pod™.
5. **Caution: Risk of Drowning!** Supervise the athlete continuously to ensure that the head does not go underwater.
6. Always have a person monitoring and holding the athlete's head and pillow to ensure a clear airway.
7. **Caution: Risk of Electrocutation!** Remove any electrical equipment from the area before using the Polar Life Pod™.