

# DELUXE HEAD WRAP & FASHION COOLING HEAD WRAP

For versatile, discreet, and fashionable body cooling, cold therapy, and more!



## THIS WRAP INCLUDES ONE OR MORE OF THE FOLLOWING PACKS FOR CONVENIENT COOLING RELIEF:

### QUICK COOL™ COOLING PACK

Reusable pack made of biodegradable, non-toxic materials that stay soft and flexible when frozen for comfortable, short-term cooling for up to 45 minutes. Ideal for cold therapy applications or whenever a quick, convenient cool down is desired.

### COOL58® PHASE CHANGE SEGMENTED COOLING PACK

Reusable pack made of non-toxic materials that are fully-charged ("frozen") solid at 58°F (14.4°C) and provides comfortable cooling at a constant 58°F for up to 1.5 hours. Pack may be recharged in ice water in approximately 15 minutes. Great for vacations and other situations when a refrigerator is not available!



If you have any questions or concerns regarding your product, please contact customer service at 800.763.8423 or send an e-mail to [polar@polarproducts.com](mailto:polar@polarproducts.com). We will be happy to help!

### EFFECTIVE AND COMFORTABLE RELIEF

Ties comfortably around the head, neck or other targeted area to provide cooling and therapeutic relief.

### DISCREET AND LIGHTWEIGHT

Lightweight, segmented cooling pack tucks discreetly in the pocket sewn in the head wrap.

### QUALITY MATERIALS

100% cotton head wrap is designed and constructed for long-term performance. The head wrap and reusable packs will last for many years with proper care, maintenance, and storage.

### IMPROVES PERFORMANCE, SAFETY AND COMFORT

Cooling lessens the debilitating effect of heat during work, athletics and day-to-day life.

# DELUXE HEAD WRAP & FASHION COOLING HEAD WRAP



## DIRECTIONS FOR USE

Activate each cooling pack as needed according to the instructions below. When the pack is activated, place the pack in the head wrap pocket or transport in a cooler for later use.

*Note: Head wraps should be tied tightly for optimal performance.*

## COOL58® PHASE CHANGE COOLING PACK

Place the pack in a cooler of ice water or refrigerator until fully-charged (“frozen”). The pack turns solid white when it is activated and will do this in approximately 15 minutes in ice water. More time is required in the refrigerator. When ready to use, simply remove the pack from the ice water or refrigerator, and insert in the head wrap pocket.

## QUICK COOL™ SOFT AND FLEXIBLE WATER-BASED COOLING PACK

Store the pack in a freezer until ready to use. Pack will remain soft and flexible when frozen. When ready to use, simply remove from the freezer and insert in the head wrap pocket.

## CARE AND STORAGE

Hand wash the head wrap in cool water with a mild detergent. Hang in a well-ventilated area to dry. Cooling packs may be wiped clean with a damp cloth.

*Note: Store packs in the refrigerator or a cool, dry environment when not in use.*

## NATURAL, DRUG-FREE COLD THERAPY

In addition to keeping you cool, our head wraps offer cold therapy when used with the Quick Cool™ Pack! Cold therapy offers effective, drug-free pain relief for bruises, muscle aches, swelling, headaches, chronic pain and much more!

*Caution: Use only the Quick Cool™ packs for cold therapy applications. Always layer insulating fabric between the pack and the skin. Cold therapy used for injuries should be limited to 20 minute intervals. Consult your health care professional.*

## WE ALSO RECOMMEND:



ADJUSTABLE COOLING VESTS



COOLING HATS AND HEADGEAR



FASHION COOLING VESTS



MOIST HEAT THERAPY

## WARNINGS AND CAUTIONS

*Consult your health care professional prior to using this or any cooling products for a medical condition. Polar Products Inc. will not be liable for injuries or damages resulting from misuse or misapplication of our products.*

