



Lithium Ion Battery Pack Instructions

Battery Pack comes with its charger. It should arrive fully charged.

The battery pack has two cords. One is labeled "To Cooler" and the other "To Charger." Use the one cord to plug into the cooler to run the unit off the battery pack and use the other cord to connect to the included charger for charging purposes only. ***Only one cord is in use at any time.***

To charge battery pack, plug in the cord labeled "To Charger" into the charger and plug into the wall. When unit is charging a red light is shown. Once battery pack is charged, a green light will show. The battery pack will run the unit for approximately 6 hours.