

Cool Comfort[®]

COOLING WRAPS AND INSERTS

Hybrid evaporative cooling technology combines the advantages of frozen water-based cooling and water-activated cooling.

Cool Comfort[®] Wraps and Inserts will cool for many hours when activated with water. Special, non-toxic polymers absorb water and cool through evaporation for long periods of time.

Cool Comfort[®] products can also be frozen (after being allowed time to dry) or refrigerated for a higher level of cooling.

Visit PolarProducts.com to see the many other body cooling and therapy products available, or call us for a complete catalog.

POLAR PRODUCTS
polarproducts.com
800-763-8423

ACTIVATION INSTRUCTIONS

Submerge in cool water for 8 to 10 minutes. Remove and gently distribute the polymer throughout the cooling wrap or insert. Re-submerge in cool water until the product reaches the level of hydration you desire. The amount of water absorbed, and therefore the weight, may be controlled to your desired level.

OPTIMAL COMFORT:

After activation in water, gently squeeze and towel dry. The product may also be hung in a well-ventilated area and allowed to dry to the touch while still retaining the water in the beads. Once dry, the wrap may be put in the freezer or refrigerator for a higher level of cooling.

CARE INSTRUCTIONS:

Hand wash in cool water with a mild detergent. Hang in a well-ventilated area to dry. Allow the product to dry completely before storage.

POLAR PRODUCTS
polarproducts.com
800-763-8423