

Water Cooling Options

Many of our customers find using **frozen water bottles** in their Active Ice® System to be convenient, easy, inexpensive, and effective. Six (6) frozen ml (16.9 FL oz) bottles will fit nicely in our 9 Qt. cooling reservoir. The frozen water bottles last a long time and there is no need to empty water from the system after each use. Just remove the bottles and re-freeze.

You can also use **frozen plastic containers of water**. The food storage containers we use holds 1 Qt. of water. Remove the ice block from the container and run under hot water to loosen the block) and put as many as will fit in the cooling reservoir.

But, ice of any form or size will work!

Just remember to use plenty of ice and to empty the excess water down to the water fill line sticker before reuse. Excess water shortens the cooling time.

Note: Always leave enough water in the reservoir to cover the pump. Make sure your water levels are at the water fill line.

