

# Polar Deluxe Torso Cooling Vests

**Vest is compatible with both Cool58® (58 °F /14.5 °C) phase change packs and Kool Max® high-energy, water-based packs.**

Included with this order (see checked box):

**Vest Type:**

- Secrets Torso Vest Wrap
- Slim Torso Cooling Vest
- Slim Torso Cooling Vest plus
- Full Torso Cooling Vest
- Other

**Kool Max® water-based 4.5" x 6" packs.** These packs are activated in a freezer and provide the highest cooling energy available.

**Cool58® phase change 4.5" x 6" packs.** These packs may be activated in a cooler of ice water, a refrigerator, or a freezer.



Secrets Vest Wrap

## Advantages of the Polar Deluxe Torso Vests

Discreet vests may be worn underneath clothing. Our torso design allows for maximum freedom of movement. Pick the size that best fits your body type, size and planned use! These vests are excellent for pre-cooling, post-cooling and cooling during an activity.

## Advantages of the Kool Max® and Cool58® cooling packs

### QUALITY MATERIALS

Polar's cooling vests are designed and constructed for long term performance. They will last for many years with proper care, maintenance and storage.

### HIGHEST COOLING ENERGY

Reusable Kool Max® water-based technology provides the highest cooling energy of any cooling pack. Kool Max® requires the use of a freezer, but the packs return this energy in cooling. It's science!

### CONVENIENT ACTIVATION

Reusable Cool58® packs can be activated in ice water, a refrigerator, or a freezer. Simply submerge your packs in ice water for up to 20 minutes or place in a refrigerator or freezer until they turn solid white, and you are ready to go!

### IMPROVES PERFORMANCE, SAFETY AND COMFORT

Cooling lessens the debilitating effects of heat during work, athletics, and day-to-day life.

### EXTENDED COOLING RELIEF

Kool Max® packs provide cooling for up to 4 hours in any climate; Cool58® packs provide cooling for up to 3 hours in any climate. Cooling time varies depending on activity, body metabolism, and environment.



Slim Torso Vest



Slim Plus Torso Vest



Torso Vest



# POLAR PRODUCTS

polarproducts.com

800-763-8423

***“I bought the secrets torso vest to use in Honduras on a medical mission. It gets very hot and humid there! I no longer have to pour ice water on my head to survive the heat. I also tried the product in Tennessee and I'm very happy with it. Thank you for this product!”***

S.G. – Honduras and Tennessee

***“The torso vest helped immensely! I went to my grandson's baseball game and had no trouble climbing a long, steep gravel path after an hour-and-a-half game, in humid weather! I have been able to enjoy many summer activities that I normally couldn't do without the torso vest!”***

M.B. – Strongsville, OH

## Activation and Use Instructions

***If you have any questions or concerns regarding your vest or packs, please contact our Customer Service at 1.800.763.8423 or send an e-mail to [polar@polarproducts.com](mailto:polar@polarproducts.com). We will be happy to help!***

### USING THE KOOL MAX® COOLING PACKS

Our reusable Kool Max® Cooling Packs are water-based, non-toxic and non-flammable. Place the packs flat into a freezer until frozen hard. For best results, store the packs in the freezer, so they're ready when you are. When ready to use, simply remove the packs from the freezer and place in the insulated vest or wrap pockets. Frozen packs may be transported or stored in a cooler for later use.

### USING THE COOL58® COOLING PACKS

Our reusable Cool58® Cooling Packs are made from a non-toxic liquid that will “freeze” at 58° F (14.5° C). Place the packs flat into a freezer, refrigerator, or bucket of ice water until frozen hard. The packs turn solid white when they're ready. When ready to use, simply remove the packs from the ice water, refrigerator, or freezer and place in the insulated pockets in the vest or wrap.

### WEARING THE VEST

The vest should fit snug to the body for optimal performance. The vest provides adjustable fit through the use of hook and loop elastic straps. Simply place the garment on your body and adjust for a snug fit.

### MAINTENANCE AND CARE

**Vest:** Hand wash in cool water with a mild detergent. Hang in a well-ventilated area to dry.

**Cooling Packs:** Packs may be wiped clean with a damp cloth. **Note: Store packs in freezer or cool, dry environment when not in use.**

### COOLING DURATION

The length of time Cool58® and Kool Max® cooling vests will cool depends on many factors, including the activity and body metabolism of the person using the items and the temperature of the environment. The cooling potential may also be adjusted by wearing one or more layers of clothing between the body and the garment. **Note: One or more layers of insulating fabric should always be worn between the skin and the cooling pack when using the packs from the freezer.**

### WE ALSO RECOMMEND:



**Cooling Fashion Scarves and Hats**



**Cooling Vests**



**Migraine and Headache Pain Relief Kit**

See our complete product line at [polarproducts.com](http://polarproducts.com)

### WARNINGS AND CAUTIONS

**Consult your health care professional prior to using this or any cooling products for a medical condition.** Kool Max® and Cool58® cooling packs are designed for the specific objective of cooling the body. Polar Products recommends that all cooling garments using Kool Max® packs be worn over clothing or other fabric layers to regulate the garments' cooling potential. The amount of layering required to ensure the highest level of cooling and comfort will vary from person to person and is strictly the responsibility of each individual user.

**Polar Products Inc. will not be liable for injuries or damages resulting from misuse or misapplication of our products.**