

Deluxe Cooling Wraps and Accessories

These versatile cooling wraps and accessories can be used in a variety of ways to provide discreet and effective cooling.

Includes plush wrap or accessory constructed with a vapor barrier.

- KMNC COOLING NECK BAND**
Cool the major arteries in the neck and help cool the body!
- KMNCEX COOLING NECK AND UPPER SPINE WRAP**
- KMSP COOLING SEAT**
Single insulation on one side, double on the other, choose the side most comfortable for you!
- KML COOLING LIMB AND GROIN WRAP WITH ELASTIC BELT**
- OTHER**

With:

- QUICK COOL™ COOLING PACK**
Reusable pack made of biodegradable, non-toxic materials **stays soft and flexible** when frozen for comfortable, short-term cooling for **up to 45 minutes**. Ideal for cold therapy applications or whenever a quick, convenient cool down is desired.
- KOOL MAX® COOLING PACK**
Reusable pack made of biodegradable, non-toxic materials **freezes solid** to provide longer term cooling for **up to 2 hours**. Pack requires access to a freezer and time to freeze but offers the highest level of cooling of any cooling pack.
- COOL58® PHASE CHANGE COOLING PACK**
Reusable pack made of non-toxic materials **freezes solid at 58 degrees Fahrenheit** (14.4° Celsius) to provide comfortable cooling at a constant 58° F for **up to 1.5 hours**. Pack may be recharged in ice water in approximately 15 minutes. Great for vacations and other situations when a freezer is not available (pack can be activated in a cooler of ice water or refrigerator!).



Advantages of the Deluxe Cooling Wraps and Accessories:

EFFECTIVE AND COMFORTABLE RELIEF: Our Polar wraps and accessories are comfortable and easy to use.

DISCREET AND LIGHTWEIGHT: Lightweight cooling packs tuck discreetly into insulated pockets sewn into the wrap or accessory.

QUALITY MATERIALS: The cooling wraps and accessories are designed and constructed for long-term performance. Wrap and reusable packs will last for many years with proper care, maintenance, and storage.

IMPROVES PERFORMANCE, SAFETY AND COMFORT: Cooling lessens the debilitating effect of heat during work, athletics and day-to-day life.



POLAR PRODUCTS

polarproducts.com

800-763-8423

Deluxe Cooling Wraps and Accessories

If you have any questions or concerns regarding your product, please contact our Customer Service at 800.763.8423 or send an e-mail to polar@polarproducts.com. We will be happy to help!

DIRECTIONS FOR USE

Activate cooling packs as needed according to the instructions below. When the packs are activated, place packs in the wrap or accessory pocket or transport in a cooler for later use.

Note: *Wrap or accessory should fit snug to the body for optimal performance.*

Kool Max® Water-Based Cooling Packs

Place packs in a freezer until frozen hard. For best results, store packs in the freezer so it's ready when you are. When ready to use, simply remove from the freezer and insert in wrap or accessory pocket.

Quick Cool™ Soft and Flexible Water-Based Cooling Packs

Store the packs in a freezer until ready to use. Note: Packs will remain soft and flexible when frozen. When ready to use, simply remove from the freezer and insert in wrap or accessory pocket.

Cool 58® Phase Change Cooling Packs

Place packs in a cooler of ice water, refrigerator, or freezer until frozen hard. The packs turn solid white when activated and will do this in approximately 15 minutes in ice water. Longer time is required in the refrigerator or freezer. When ready to use, simply remove the packs from the ice water, refrigerator, or freezer and insert in wrap or accessory pocket.

Caution: *one or more layers of insulating fabric should always be worn between the skin and the Kool Max® cooling packs.*

CARE AND STORAGE

Hand wash the wrap or accessory in cool water with a mild detergent. Hang in a well-ventilated area to dry. Cooling packs may be wiped clean with a damp cloth.

Note: *Store packs in the freezer or cool, dry environment when not in use.*

NATURAL, DRUG-FREE COLD THERAPY

In addition to keeping you cool, the Deluxe Cooling Wraps and Accessories offer cold therapy when used with the Quick Cool™ Pack! Cold therapy offers effective drug-free pain relief for bruises, muscle aches, swelling, headaches, chronic pain and much more! The wrap or accessory can be used to position the pack on the target area. **Caution:** *Use only the Quick Cool™ packs - not the Kool Max® packs - for cold therapy applications. Always layer insulating fabric between the pack and the skin. Cold therapy used for injuries should be limited to 20 minute intervals. Consult your health care professional.*

WARNINGS AND CAUTIONS

Consult your health care professional prior to using this or any cooling products for a medical condition. Kool Max® and Cool58® cooling packs are designed for the specific objective of cooling the body. Polar Products recommends that all cooling garments using Kool Max® packs be worn over clothing or other fabric layers to regulate the garments' cooling potential. The amount of layering required to ensure the highest level of cooling and comfort will vary from person to person and is strictly the responsibility of each individual user.

Polar Products Inc. will not be liable for injuries or damages resulting from misuse or misapplication of our products.

WE ALSO RECOMMEND:



Cooling Fashion Scarves and Hats



Cooling Vests



Migraine and Headache Pain Relief Kit

See our complete product line at polarproducts.com



POLAR PRODUCTS

800-763-8423
polarproducts.com